

CARING FOR YOUR LOUPES + LIGHTS

tips from Andau Medical for looking after your investment

For full instructions and guidance, please refer to the Instructions for Use (IFU) included with your product.



USE A DISINFECTANT SPRAY

Spray the alcohol directly onto a soft cloth and then use the damp cloth to wipe your loupes and headlight system. Optim 1 Disinfectant can be used on the Loupes & Lights. Using any other form of disinfectant will void the product warranty.

DO NOT SUBMERGE YOUR LOUPES

Please DO NOT spray loupes directly or submerge the oculars in any liquid or ultrasonic cleaner. This could allow water to penetrate the seal around the oculars.



USE A POLISHING CLOTH

After the disinfection process is over, use an optical polishing cloth (included) or a pre-packaged optical wipe to remove any streaks.

CLEAN REGULARLY

Using canned air to blow off your products will help with the cleaning process. Environmental dust and other debris can negatively affect the performance of your loupes and lights.



POWER OFF + STORE

Do not leave a fully charged battery plugged in. This can cause damage to a Lithium-Ion battery, including diminished capacity. Best practice is to deplete battery before charging, and store in its case. Turn off the power and store your products in the case when not in use.

Keep magnetic connecting prongs on the butterfly battery free of debris for best connectivity. Use a 70% alcohol wipe to clean.



ANDAUMEDICAL